



Affinis Inverse Metaglène CP

Instructions for technical surgical assistance

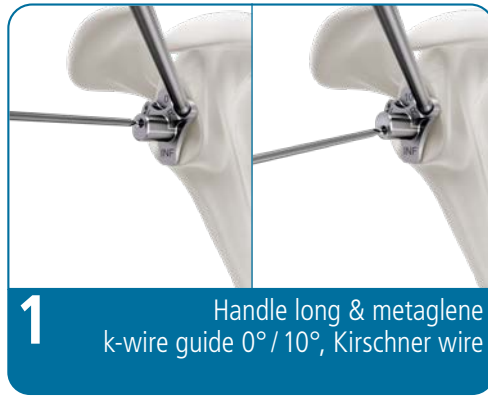


1 Handle long & metaglène k-wire guide 0° / 10°, Kirschner wire

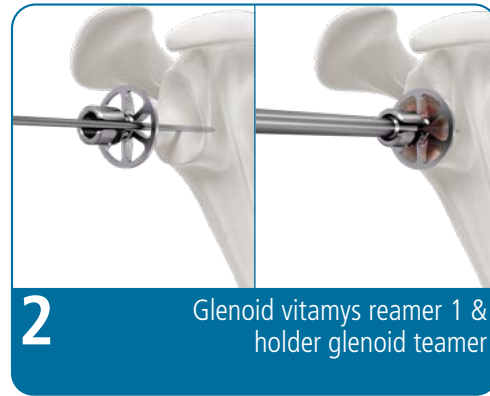


2 Glenoid vitamys reamer 1 & holder glenoid teamer



3 Glenoid Reamer 42

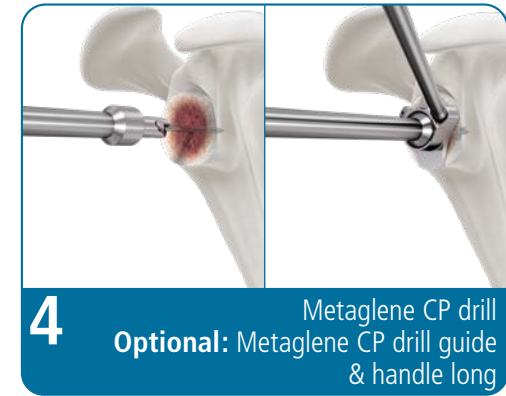

4 Optional: Metaglène CP drill guide & handle long

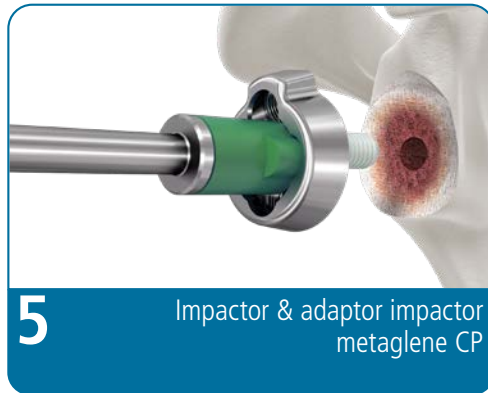

5 Impactor & adaptor impactor metaglène CP

The sequence of steps 6–8 are to be repeated for inferior / superior screw implantation.




6 Drill guide 3.0 & drill bit 3.0 with quick coupling




7 Depth gauge LC




8 Screwdriver T20

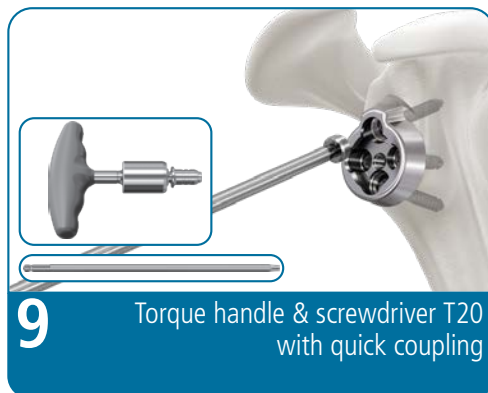

9 Torque handle & screwdriver T20 with quick coupling

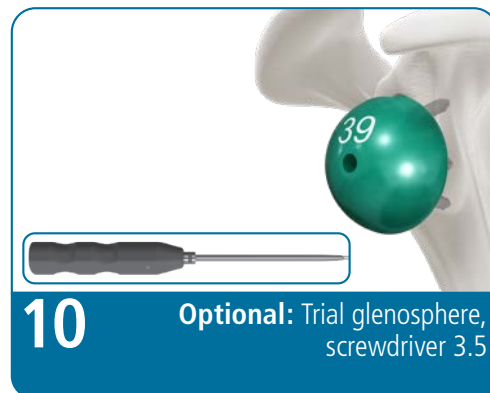

10 Optional: Trial glenosphere, screwdriver 3.5

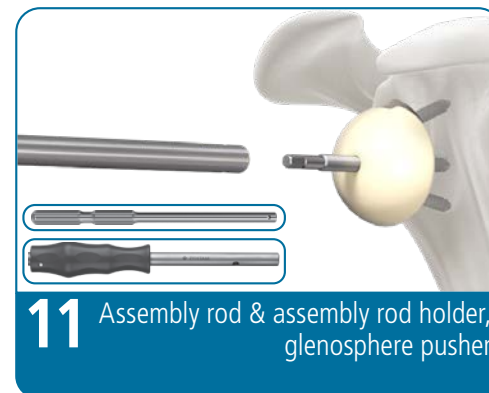

11 Assembly rod & assembly rod holder, glenosphere pusher



12 Screwdriver 3.5

These instructions are intended for technical surgical assistance and describe the sequence in which the instruments are used. **They are not a description of the surgical technique.** The surgical procedure is described in detail in the document Affinis Inverse metaglène CP surgical technique (item no. 336.020.041).